

Model: **21020**

Manufactured for Zewa, Inc.
12960 Commerce Lakes Drive # 29
Fort Myers, FL 33913
USA
www.zewa.com

Toll Free Customer Service:
1-888-993-3592
warranty@zewa.com



Instruction Manual

SpaBuddy Massage Belt

Model #: 21020



Please keep this instruction manual in a safe place.

Table of Contents

Introduction.....	1
Important Warnings.....	1-2
Precautions.....	3
Package Contents.....	3-4
Control Unit.....	4
Preparation.....	5
Using the Massage Belt.....	5-7
Possible Placements.....	7-8
Care and Maintenance.....	8-9
Technical Specifications.....	9

Introduction

Thank you for purchasing a quality Zewa product. The materials and components of this product have specifically been designed for your device. The use of this device will be described within the following pages. Please read all instructions carefully before use.

Important Warnings

Important Safeguards

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

WARNING

To reduce the risk of burns, electric shock, fire or injury to persons:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by Zewa; specifically any attachments not provided with the unit.

- NEVER operate this appliance if it has a damaged cord or plug, it is not working properly, if it has been dropped or damaged, or dropped into water, return it to Zewa's Service department for examination and repair.
- Keep cord away from heated surfaces.
- NEVER use while sleeping or falling asleep.
- NEVER drop or insert any objects into openings.
- Always unplug the appliance from the electrical outlet immediately after using and before cleaning
- DO NOT use while bathing or in shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink.
- Do NOT place in or drop into water or liquid.
- NEVER use pins or other metallic fasteners with this appliance.
- Keep dry – DO NOT operate in wet or moist condition.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT carry this appliance by power cord or use cord as handle.
- To disconnect turn all controls to the "OFF" position, then remove the plug from the outlet.
- This appliance is designed for personal use only.
- Do NOT use outdoors.
- DO NOT crush. Avoid sharp folds.
- Use heated surfaces carefully. May cause serious burns.

Precautions

- This product is not intended for medical use. It is intended to provide a gentle massage.
- Consult your doctor prior to using this product if
 - You are pregnant
 - You have a pacemaker
 - You have any concerns regarding your health
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for a medical problem.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capability to operate controls.
- This unit should not be used by Children or invalids without adult supervision.

Package Contents

Before using the device, make sure that you have all components.

1x Massage Belt	1x Car Adaptor
1x Belt Extension	1x Instruction manual
1x AC Adaptor	1x Warranty Card



1



Belt Extension


2



3



4

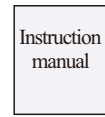

Instruction manual

5




Warranty Card

6

Control Unit

- 1 LCD Display
- 2 Heat indicator light
- 3 PRG Button
- 4 Timer Button
- 5 Intensity Button
- 6 Power/Heat button

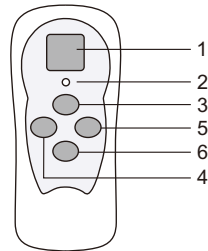

Preparation

Connect the power adaptor to the massage belt and the power outlet. If used on the back, wrap the belt around your body, if the belt is too small, the enclosed Velcro extension can be used. If used on legs or shoulder the massage belt can also be used as a massage “mat or pillow” by folding the 2 “wings” to the back of the massage belt. The Velcro will hold those 2 parts folded on the back of the massage belt.

Using the Massage Belt

Once the power cord is connected the display will show -- indicating that the unit is plugged in.

Turning it ON and OFF

To turn the unit ON, push the Power button. To turn the unit OFF, push the power button for 3 seconds until it turns OFF.

Note: When the unit is turned ON the display will show 15 indicating that the timer is set to 15 minutes. This number will change indicating the time left before the unit turns off. Please refer to “Timer” for more information.

Note: Always unplug the AC adaptor if product is not in use! This is the ONLY way to completely

turn OFF the massage belt.

Programs :

This massage belt features 7 different massage programs. To change to a different program, simply push the PRG button. Pushing the button once will display the current program number, pushing it again will switch to the next program.

Note: Program 0 (default when turned ON) circulates through all 7 programs.

Note: There is a 10 second time limit to change from 1 program to the next. After 10 seconds the unit will switch back to displaying the time.

Intensity :

There are 3 intensity levels. To change the intensity, push the intensity button. Pushing the button once will display the current intensity level, pushing it again will switch to the next level.

Note: There is a 10 second time limit to change from 1 intensity level to the next. After 10 seconds the unit will switch back to displaying the time.

Heat function:

When the unit is initially turned ON, the heat function is turned ON as well. To turn the heat OFF, push the Power/Heat button. To turn the heat ON again, simply push the Power/Heat button again.

A red indicator light (located below the display) will indicate if the heat function is ON or OFF. The heat

function will reach up to 113° Fahrenheit (45° degrees Celsius).

Please note this is NOT a heat wrap the temperature is limited to 113° Fahrenheit (45° degrees Celsius). Higher heat could damage the motors.

Timer:

The unit has 4 different timer functions. When the unit is initially turned ON it will automatically start at 15 minutes by showing 15 on the display. To change the Timer, push the TIME button.

- 1) 5 Minutes
- 2) 15 Minutes
- 3) 25 Minutes
- 4) FF (continuous, unit has to be turned off manually)

The display will show time remaining until the unit turns OFF.

Possible Placements





Care and Maintenance

Storage:

Before storing the unit, make sure that it is unplugged from the power source. Avoid contact with sharp edges or pointed objects which might cut or puncture the product.

Do NOT wrap the massage belt too tight, it should be wrapped using its original shape. Do

NOT wrap the power cord around the unit.

Cleaning:

Unplug the unit and allow it to cool before cleaning. Spot cleaning only with soft, slightly damp sponge or cloth. Never saturate the fabric. Let completely dry before using the unit again.

- Do NOT immerse in any liquid to clean
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass / furniture polish or paint thinner to clean.
- Do not attempt to repair. There are no user serviceable parts.

Technical Specifications

Model No.	21020
Weight:	Approx. 1.4lbs.
Vibrating Motors:	4
Max temperature	113° Fahrenheit (45° degrees Celsius)
Rated Voltage (AC)	110V
Power Adaptor Output	12V
Frequency:	50-60Hz
Watts:	24